

# Pure C&G AX

#### Data Sheet

<sup>Made for</sup> **€** iPhone | iPad | iPod 7AX 5AX 3AX 2AX 1AX DAX



#### S-Receiver

- 46 dB / 110 dB SPL (2 ccm coupler)
- 56 dB / 120 dB SPL (Ear simulator)

#### **M-Receiver**

- 60 dB / 119 dB SPL (2 ccm coupler)
- 70 dB / 129 dB SPL (Ear simulator)

#### **P-Receiver**

- 65 dB / 122 dB SPL (2 ccm coupler)
- 75 dB / 131 dB SPL (Ear simulator)

#### **HP-Receiver**

- 75 dB / 131 dB SPL (2 ccm coupler)
- 83 dB / 138 dB SPL (Ear simulator)

## Pure C&G AX | Technical Data

Туре	S-Receiver		M-Receiver		
	2 ccm coupler	Ear simulator	2 ccm coupler	Ear simulator	
Output sound pressure level					
OSPL 90 at 1.6 kHz	-	110 dB SPL	_	123 dB SPL	
OSPL 90 (peak)	110 dB SPL	120 dB SPL	119 dB SPL	129 dB SPL	
HFA OSPL 90	102 dB SPL	-	115 dB SPL	-	
Gain					
FOG at 1.6 kHz	-	44 dB	_	58 dB	
FOG (peak)	46 dB	56 dB	60 dB	70 dB	
HFA FOG	38 dB	-	51 dB	_	
Reference test gain	25 dB	35 dB	38 dB	48 dB	
Frequency, noise and directivity					
Frequency range 7AX 5AX / 3AX / 2AX / 1AX	100 – 10000 Hz 100 – 8200 Hz	100 – 10000 Hz 100 – 8300 Hz	100 – 9500 Hz 100 – 8200 Hz	100 – 10000 Hz 100 – 8300 Hz	
Equivalent input noise	16 dB SPL	19 dB SPL	16 dB SPL	19 dB SPL	
Total harmonic distortion at 500 / 800 / 1600 / 3200 Hz	1/1/1/1%	1 / 1 / 2 / – %	1/1/1/1%	2/2/3/-%	
Tinnitus Function broadband	65 dB SPL	-	70 dB SPL	_	
AI-DI	4.0 dB		4.0 dB		
Latency	< 15 ms		< 15 ms		
Inductive coil sensitivity					
MASL (1 mA/m) at 1.6 kHz	-	-	_	_	
HFA MASL (1 mA/m)	-	-	_	-	
HFA SPLITS (left/right)	_	-	_	_	
RSETS (left/right)	-	-	_	-	
HFA SPLIV	-	-	_	-	
Battery					
Battery runtime (without streaming)	up to 28 h		up to 28 h		
Battery runtime (incl. 5 h streaming)	up to 24 h		up to 24 h		
Cellphone Compatibility					
Microphone mode	0.65 – 0.96 GHz 1.4 – 2.7 GHz		0.65 – 0.96 GHz 1.4 – 2.7 GHz		
Telecoil mode	_			_	


Please find additional information to the values on page "Further information".


## Pure C&G AX | Technical Data

Туре	P-Receiver		HP-Receiver		
	2 ccm coupler	Ear simulator	2 ccm coupler	Ear simulator	
Output sound pressure level					
OSPL 90 at 1.6 kHz	-	129 dB SPL	_	136 dB SPL	
OSPL 90 (peak)	122 dB SPL	131 dB SPL	131 dB SPL	138 dB SPL	
HFA OSPL 90	120 dB SPL	-	124 dB SPL	-	
Gain					
FOG at 1.6 kHz	-	69 dB	_	82 dB	
FOG (peak)	65 dB	75 dB	75 dB	83 dB	
HFA FOG	61 dB	-	69 dB	-	
Reference test gain	43 dB	54 dB	47 dB	61 dB	
Frequency, noise and directivity					
Frequency range 7AX 5AX / 3AX / 2AX / 1AX	100 – 7400 Hz 100 – 7400 Hz	100 – 8000 Hz 100 – 8000 Hz	100 – 7700 Hz 100 – 7700 Hz	200 – 7500 Hz 200 – 7500 Hz	
Equivalent input noise	14 dB SPL	16 dB SPL	15 dB SPL	8 dB SPL	
Total harmonic distortion at 500 / 800 / 1600 / 3200 Hz	1/2/1/1%	2/3/3/-%	1/2/1/1%	2/3/2/-%	
Tinnitus Function broadband	75 dB SPL	_	85 dB SPL	_	
AI-DI	4.0 dB		4.0 dB		
Latency	< 15 ms		< 15 ms		
Inductive coil sensitivity					
MASL (1 mA/m) at 1.6 kHz	-	-	_	_	
HFA MASL (1 mA/m)	_	_	_	_	
HFA SPLITS (left/right)	_	-	_	_	
RSETS (left/right)	_	_	_	_	
HFA SPLIV	_	-	_	_	
Battery					
Battery runtime (without streaming)	up to 28 h		up to 28 h		
Battery runtime (incl. 5 h streaming)	up to 24 h		up to 24 h		
Cellphone Compatibility					
Microphone mode	0.65 – 0.96 GHz 1.4 – 2.7 GHz		0.65 – 0.96 GHz 1.4 – 2.7 GHz		
Telecoil mode	_		-	_	


Please find additional information to the values on page "Further information".

## Pure C&G AX | Fitting Range

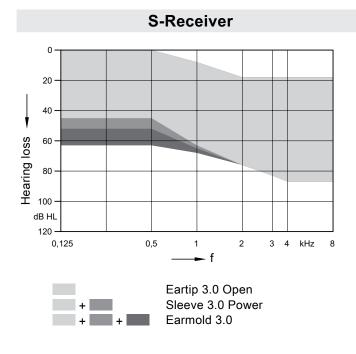


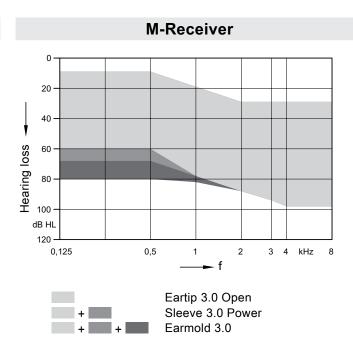

**P-Receiver** 

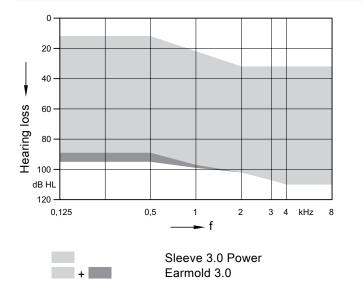


**HP-Receiver** 

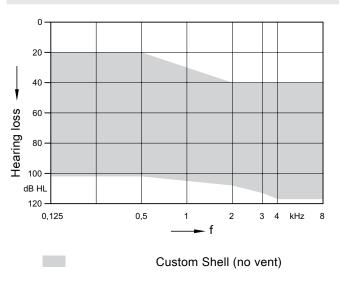


### S-Receiver (Sleeve 3.0 Power) | Basic Data

#### 2 ccm coupler Ear simulator 140 dB SPL 130 140 dB SPL 130 Max. Output Max. Output sound pressure sound pressure $L_{o}$ $L_{o}$ 120 120 level level (L<sub>1</sub> = 90 dB) (L<sub>1</sub> = 90 dB) 110 110 100 100 90 90 Full on gain Full on gain 80 80 $(L_1 = 50 \text{ dB})$ $(L_1 = 50 \text{ dB})$ 70 0.125 70 0.125 0.25 0.5 2 3 4 kHz 8 10 0.25 0.5 3 4 kHz 8 10 1 1 2 – f **-** f 140 dB SPL 130 140 dB SPL 130 Frequency **Basic acoustic** response response $L_{o}$ $L_{o}$ 120 $(L_1 = 60 \text{ dB})$ 120 $(L_1 = 60 \text{ dB})$ 110 110 ۱ 100 100 90 90 80 80 70 70 0.125 0.25 0.5 1 2 3 4 kHz 8 10 0.125 0.25 0.5 1 2 3 4 kHz 8 10 ► f **-** f


## M-Receiver (Sleeve 3.0 Power) | Basic Data

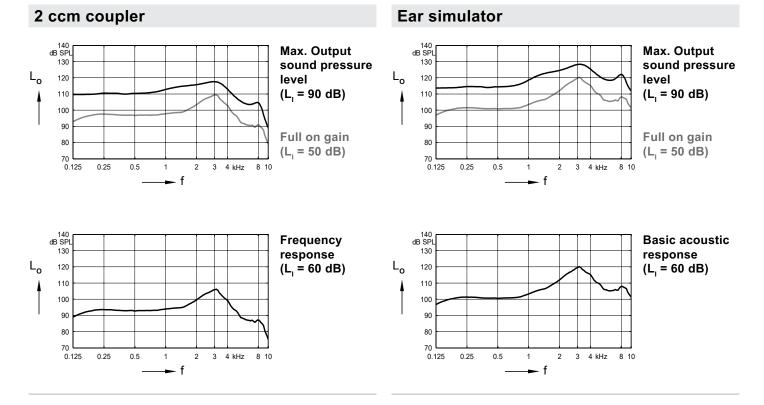

## P-Receiver (Earmold 3.0) | Basic Data

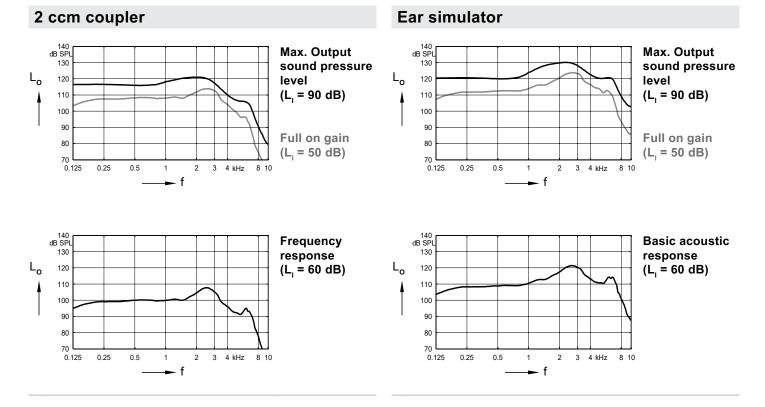

#### HP-Receiver (Custom Shell) | Basic Data

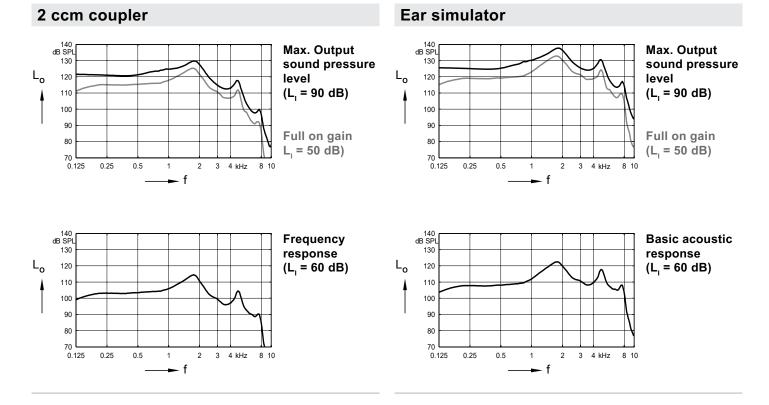

## Pure C&G AX | Features and Accessories

	7AX	5AX	3AX	2AX	1AX
Dynamic Soundscape Processing 2.0					
Augmented Focus	1	$\checkmark$	1	1	$\checkmark$
Acoustic Sensor	$\checkmark$	$\checkmark$	1	1	1
Motion Sensor	1	$\checkmark$	1	_	_
OVP 2.0 (Own Voice Processing) <sup>1)</sup>	$\checkmark$	$\checkmark$	$\checkmark$	—	—
Sound Clarity					
Signal processing (channels) / Gain&MPO (handles)	48 / 20	32 / 16	24 / 12	16 / 8	16 / 8
Hearing programs	6	6	6	4	4
Extended dynamic range	$\checkmark$	$\checkmark$	1	1	1
Speech and noise management	$\checkmark$	$\checkmark$	√	1	✓
SoundSmoothing	✓	$\checkmark$	1	1	
Feedback cancellation	$\checkmark$	$\checkmark$	√	1	1
HD Music (presets)	3	3	1	1	_
eWindScreen	1	$\checkmark$	1	1	_
Extended bandwidth	1			_	
Auto EchoShield	1		_		_
EchoShield	1	$\checkmark$			
Speech Quality					
Binaural Directionality	1	$\checkmark$	1	_	_
Wireless CROS/BICROS	1	$\checkmark$	√	1	1
Frequency compression	✓	$\checkmark$	1	1	1
Spatial SpeechFocus <sup>1) 2)</sup>	$\checkmark$	$\checkmark$	_		_
Wearer Interaction					
Signia Assistant	$\checkmark$	$\checkmark$	1	1	1
Signia App (iOS and Android)	$\checkmark$	√	√	1	1
Adaptive Streaming Volume <sup>3)</sup>	$\checkmark$	$\checkmark$	1	1	1
Spatial Configurator	$\checkmark$	√			_
Direct Streaming	$\checkmark$	$\checkmark$	$\checkmark$	$\checkmark$	$\checkmark$
Android devices (ASHA)	$\checkmark$	$\checkmark$	1	1	1
Made for iPhone   iPad   iPod	$\checkmark$	$\checkmark$	1	1	✓
HandsFree for iOS	$\checkmark$	$\checkmark$	$\checkmark$	_	_
Tinnitus	$\checkmark$	$\checkmark$	$\checkmark$	$\checkmark$	_
Notched Amplification Therapy	$\checkmark$	$\checkmark$	1	1	_
Tinnitus noise therapy signal	$\checkmark$	1	✓	1	_
Fitting	$\checkmark$	$\checkmark$	$\checkmark$	$\checkmark$	$\checkmark$
Smart Optimizer and Data Logging	$\checkmark$	$\checkmark$	✓	1	1
Acclimatization manager	$\checkmark$	√	✓	✓	✓
InSituGram	√	√	✓	✓	1
AutoFit	✓	√	✓	✓	✓
TeleCare	$\checkmark$	$\checkmark$	$\checkmark$	$\checkmark$	$\checkmark$
Remote Services	1	$\checkmark$	√	1	1
Signia App	$\checkmark$	$\checkmark$	1	1	√

<sup>1)</sup> req. bilateral fitting
<sup>2)</sup> for 5AX, right/left directionality available only in Stroll Program and via the Spatial Configurator

highest feature performance

✓ available — not available O optional

3) streaming only

## Pure C&G AX | Features and Accessories

Type specific features	
Ingress Protection Rating	IP68
Charging contacts	✓
Battery size	
Battery door on/off function	
Nanocoated housing	✓
e2e wireless 4.0	✓
User controls coupling via e2e	✓
Wireless programming	√
Instrument configurations	
Flat cover	_
Rotary volume control	
Push button	
Rocker switch	✓
Color conversion kit	0
Color conversion kit with T-Coil	
T-Coil	
Battery door – tamper proof	
Battery door – child lock	_
Programming accessories	
ConnexxAir / ConnexxLink	<u> </u>
Noahlink Wireless	mandatory
Programming adapter / cable	
Accessories	
miniPocket	0
StreamLine TV	0
StreamLine Mic	0
Charger RIC / D&C Charger RIC / Portable Charger RIC	mandatory
Small earhook —	
CROS Pure C&G AX	0
CROS Pure 312 AX	0
CROS Styletto AX	

✓ available — not available O optional

## Pure C&G AX | Further information

#### Abbreviations

The following abbreviations are used in this datasheet:

SPL	Sound Pressure Level
OSPL	Output Sound Pressure Level
HFA	High Frequency Average
FOG	Full-On Gain
MASL	Magneto Acoustical Sensitivity Level
SPLITS	Coupler SPL for an Inductive Telephone Simulator
RSETS	Relative Simulated Equivalent Telephone Sensitivity
SPLIV	SPL In a Vertical magnetic field
AI-DI	Articulation Index - Directivity Index
IRIL	Input Related Interference Level
RTF	Reference Test Frequency
ASHA	Audio streaming for hearing aids

#### Standards and additional information

- All measurements with the 2 ccm coupler were performed according to ANSI S3.22-2014 and IEC 60118-0:2015 if applicable.
- All measurements with an ear simulator were performed according to IEC 60118-0:1983 + A1:1994 and to DIN 45605 (frequency range) if applicable.
- All Cellphone Compatibility measurements were performed according to IEC 60118-13:2019, EN IEC 60118-13:2020 and ANSI C63.19-2019.
- Cellphone Compatibility definition: It is expected that the hearing aid user can effectively use a compliant wireless device held in a talking position at the ear. Maximum achievable Cellphone Compatibility range: 0.65–0.96 GHz and 1.4–2.7 GHz.
- Curves and figures representing FOG are measured with 20 dB reduction and 70 dB SPL input level.
- Figures representing Equivalent Input Noise incorporate a moderate expansion.
- Tinnitus noiser measurement conditions: all tinnitus single frequency sliders in max position, master volume slider in default position (0 dB) and local volume control in default position.
- Inductive coil sensitivity values, inductive response curves and T ratings apply for instruments with telecoil only.
- The current consumption is measured in reference test setting (RTS) according to the applicable standards. Due to the settling behaviour of hearing aids supporting RF (radio frequency), the battery current is measured 3 minutes after turning on (note: no pairing).
- The battery runtime is based on first fit settings using 60 % of the fitting range and an ISTS (International Speech Test Signal) input signal at 65 dB SPL (note: pairing established). The actual battery runtime is determined by battery quality, hearing loss, sound environment, usage and activated feature set. Regarding RF usage, Bluetooth audio streaming from phone to hearing aid and from hearing aid to phone are considered.
- Extended bandwidth up to 12 kHz for 7AX devices only.
- The following acoustic connections/ear pieces were used:
  - S-Receiver Unit and M-Receiver Unit: Sleeve 3.0 Power
  - P-Receiver Unit: Earmold 3.0
  - HP-Receiver Unit: Custom Shell

#### Special note for instruments with built-in lithium-ion rechargeable battery

The runtime of all lithium-ion rechargeable batteries reduces over time. The estimates are based on fresh lithium-ion rechargeable battery capacity. Under normal operating conditions, the battery will retain up to 80 % of its initial capacity after 3 years of use. Please note that battery performance will vary depending on individual usage patterns and environmental conditions.

<sup>Made for</sup> **€** iPhone | iPad | iPod "Made for iPhone", "Made for iPad", and "Made for iPod" mean that an electronic accessory has been designed to connect specifically to iPhone, iPad, or iPod, respectively, and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this accessory with iPhone, iPad, or iPod may affect wireless performance.

The information in this document contains general descriptions of the technical options available, which do not always have to be present in individual cases and are subject to change without prior notice. The required features should therefore be specified in each individual case at the time of conclusion of the respective contract.

Order No. 04605-99T04-7600 www.wsaud.com © 01.2023, WSAUD A/S All rights reserved

Subject to change without prior notice